



MGC
DIAGNOSTICS®

Informational Product Bulletin

dba Medical Graphics Corporation
www.mgcdiagnostics.com

350 Oak Grove Parkway Tel (651-484-4874)
St. Paul, Minnesota 55127-8599 Fax (651-484-8941)

Applicable To:

MGC Employee Distributor Customer

Product(s): preVent Mask

Date: August 3, 2012

Subject: Silicone Coupler Cleaning & Disinfection

PURPOSE: To provide information for cleaning and disinfection of the Silicone Coupler, part numbers 670010-003 and 531033-001.

PROCEDURE /INSTRUCTIONS:

Cleaning

1. Remove the Silicone Coupler from the preVent™ Mask.
2. Rinse the Silicone Coupler with warm tap water for 2 minutes using a soft cleaning brush to loosen and remove any foreign matter.

Disinfection

1. Soak the Silicone Coupler in a solution of 10% household bleach and water for 60 minutes
OR soak in a 2.4% glutaraldehyde solution for 45 minutes.
2. Rinse the Silicone Coupler for five minutes using tap water.

This product is made from silicone. Other disinfection methods must be verified safe for silicone before using.

Please visit our website www.mgcdiagnostics.com for additional product support via the answer book.