



MGC
DIAGNOSTICS®

Informational Product Bulletin

dba Medical Graphics Corporation
www.mgcdiagnostics.com

350 Oak Grove Parkway Tel (651-484-4874)
St. Paul, Minnesota 55127-8599 Fax (651-484-8941)

Applicable To:

MGC Employee Distributor Customer

Product(s): preVent™ Mask

Date: August 3, 2012

Subject: Material Handling and Cleaning Instructions

BACKGROUND: This product is intended for single subject use only.

PURPOSE: To provide material handling and cleaning instructions.

PROCEDURE /INSTRUCTIONS:

Material Handling:

The preVent™ Mask is manufactured from neoprene. In rare cases neoprene has been known to cause dermatological reactions. Two known reactions are *allergic contact dermatitis (ACD)* and *miliaria rubra (prickly heat)*. It is recommended that users be screened for a history of dermatological reactions to neoprene before wearing of the preVent™ Mask.

Cleaning Instructions:

1. Carefully separate the neoprene preVent™ Mask from the silicon coupler and preVent™ pneumotach.
2. Wash the facemask separately in warm water with a mild detergent. Rinse with cool water and allow it to air dry.
3. After multiple washing cycles, mask material may degrade and should be replaced.

Helpful Hint:

Install the silicone coupler through the inside of the preVent™ Mask with larger end of coupler on outside of mask.

Please visit our website www.mgcdiagnostics.com for additional product support via the answer book.